

Symphony® Trolley (#6007092)

Parts Guide


- A. 5 Wheel base
- B. Tube
- C. Upper Base
- D. Threaded rod
- E. Coupler
- F. Retaining ring
- G. Nylok™ nut
- H. Allen wrench
- I. Washer

Replacement Symphony Trolley Parts

Upper Base,
Item #8000682
(contains part# C)

5 Wheel Base Kit,
Item #6007090
(contains part# A, E, F, & G)

Repair Kit,
Item #6007116
(contains part# F, G, & I)

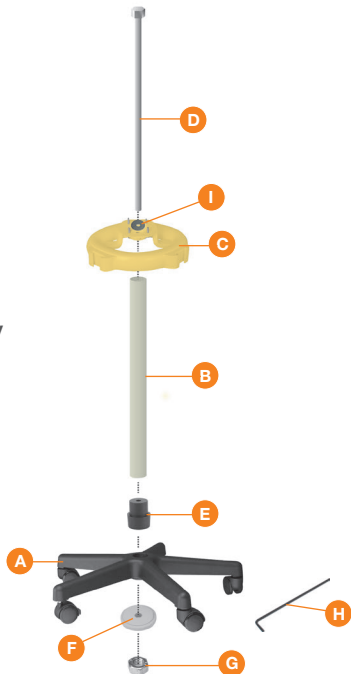

For more information, call Medela
Customer Service at 888-MEDELA-8
or visit www.medela.com



www.medela.com

Medela, Inc., 1101 Corporate Drive, McHenry, IL 60050
Phone: (800) 435-8316 or (815) 363-1166 Email: customer.service@medela.com
Medela and Symphony are registered in the U.S. Patent and Trademark Office.
Nylok is a trademark of Nylok, LLC.

ART.1948384 E 0716 © 2016 Medela, Inc. 1948384 All rights reserved.

medela 


Symphony® Trolley

Assembly Instructions

*breast milk
is best*




Symphony® Trolley QuickStart Instructions

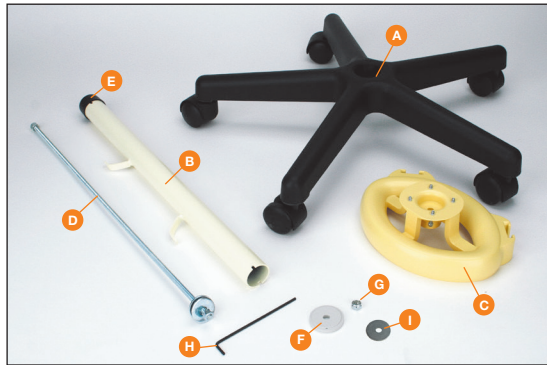


1 Verify all parts. If present, remove toothed lock washer and nut without nylon lock insert.

- A. 5 Wheel base
- B. Tube
- C. Upper base
- D. Threaded rod


- E. Coupler
- F. Retaining ring
- G. Nylok™ nut
- H. Allen wrench
- I. Washer

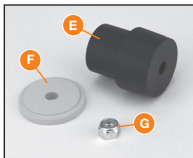

Keep Nylok™ nut, retainer ring and coupler.

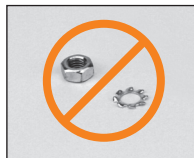


If present discard.

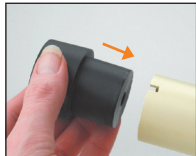


2 If not installed, insert coupler into tube.




3 Insert tube into 5 wheel base with the coupler side down.

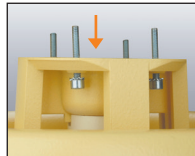


4 Place upper base on top of tube, aligning notches in platform to tube.

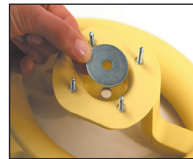

5 Place washer over tube opening.

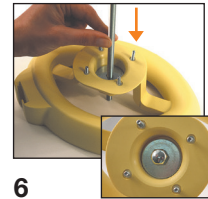

6 Insert threaded rod through tube and washer.

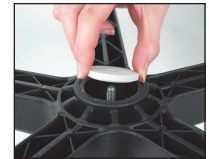

7 Insert retaining ring into 5 wheel base.



8 Start Nylok™ nut by hand.



9 Tighten rod by using 13 mm socket wrench while holding nut, which is located on bottom of trolley, with adjustable wrench. (see image #10)



10 Tighten nut firmly. 1/4 to 1/2 inch of threaded rod should extend past tightened nut.



11 Turn upright.



12 Line up screws with base of pump.



13 Mount Symphony breast pump to upper base with 3.0 mm Allen wrench.